

California Device Recall Information Sheet

Food and Drug Branch – Device Recalls

GE Medical Systems China Co., Ltd. GE Healthcare Carestation 620 A1 A2 SE, 650 A1 A2 SE, 650C A1 A2, 750 A1 A2, 750C A1 A2

Recall Date	Product Description	Recalling Firm	Recall Reason
04/29/2025	GE Healthcare Carestation 620 A1 Model/REF Number: 1012-9620-000 GE Healthcare Carestation 620 A2 Model/REF Number: 1012-9620-002 GE Healthcare Carestation 620 SE Model/REF Number: 1012-9620-012 GE Healthcare Carestation 620 A1 Model/REF Number: 1012-9620-200 GE Healthcare Carestation 650 A1 Model/REF Number: 1012-9650-000 GE Healthcare Carestation 650 A2 Model/REF Number: 1012-9650-002	GE Medical Systems China Co., Ltd.	GE HealthCare has become aware that certain Carestation 620/650/650c and 750/750c Anesthesia Delivery Systems will not provide effective ventilation in Volume Control Ventilation (VCV) mode. In these systems, effective ventilation can be achieved in Pressure Control Ventilation (PCV) or Pressure Control Ventilation Volume Guarantee (PCV-VG) modes or with Manual ventilation.

	GE Healthcare Carestation 650 SE Model/REF Number: 1012-9650-012 GE Healthcare Carestation 650 A1 Model/REF Number: 1012-9650-200 GE Healthcare Carestation 650C A1 Model/REF Number: 1012-9655-000 GE Healthcare Carestation 650C A2 Model/REF Number: 1012-9655-002 GE Healthcare Carestation 650c A1 Model/REF Number: 1012-9655-200 GE Healthcare Carestation 750 A1 Model/REF Number: 1012-9750-000 GE Healthcare Carestation 750 A2 Model/REF Number: 1012-9750-002 GE Healthcare Carestation 750c A1 Model/REF Number: 1012-9755-000 GE Healthcare Carestation 750c A2 Model/REF Number: 1012-9755-002		
--	--	--	--

Recall Class	Product Identification	Distribution	Affected Dates
I	GTIN: 00840682103985 Serial Numbers: SM623390002MA - SM624440017MA GTIN: 00840682124546 Serial Numbers: SM623510024WB - SM623490002WB GTIN: 00195278569677 Serial Numbers: SM624250018WC - SM624330023WC GTIN: 00195278439536 Serial Numbers: SM624510015MD - SM624320004MD GTIN: 00840682103947 Serial Numbers: SM724310010MA - SM724290029MA GTIN: 00840682124560 Serial Numbers: SM723010002WB - SM723310003WB GTIN: 00195278569684 Serial Numbers: SM724370002WC GTIN: 00195278439529 Serial Numbers: SM724450001MD - SM724320001MD GTIN: 00840682103954 Serial Numbers: SM823080001MA - SM824440013MA	14,253 units nationwide	March and prior

	GTIN: 00840682124539 Serial Numbers: SM823250002WB - SM823460002WB GTIN: 00195278439543 Serial Numbers: SM824450008WD - SM825020006MD GTIN: 00840682145596 Serial Numbers: SQS24500010MA - SQS24250002MA GTIN: 00840682146470 Serial Numbers: SQS23480001WB - SQS24090003WB GTIN: 00840682146425 Serial Numbers: SQT23330001WA - SQT23190009WA GTIN: 00840682146463 Serial Numbers: SQT23170001WB - SQT24020001WB		
--	---	--	--

For additional information, please visit: [FDA Website \(620 A1\)](#), [FDA Website \(620 A2\)](#), [FDA Website \(620 SE\)](#), [FDA Website \(620 A1 2\)](#), [FDA Website \(650 A1\)](#), [FDA Website \(650 A2\)](#), [FDA Website \(650 SE\)](#), [FDA Website \(650 A1 2\)](#), [FDA Website \(650C A1\)](#), [FDA Website \(650C A2\)](#), [FDA Website \(650C A1 2\)](#), [FDA Website \(750 A1\)](#), [FDA Website \(750 A2\)](#), [FDA Website \(750C A1\)](#), and [FDA Website \(750C A2\)](#).