

# Battelle CCDS™ Process

**BATTELLE**

## HEALTH CARE PROVIDER SIGN-UP PROCESS Battelle CCDS Critical Care Decontamination System™



# Battelle CCDS™ Process

## HOW IT WORKS

### Battelle CCDS Critical Care Decontamination System™

**BATTELLE**

#### Your Facility's Responsibility

- Your Battelle POC will provide you with 3-digit codes for each facility
- The 3-digit code must be written on each individual N95

- Collect all N95 respirators into a single plastic bag
- Once the plastic bag is filled, tie off the bag and put it into another plastic bag

- Clean the outside bag with disinfectant
- Shipping box must be labeled with the 3-digit code and a biohazard sticker

- You can either use a logistics provider of your choice or Battelle's preferred logistics provider

#### Battelle CCDS™ Process

- Your shipments are barcoded to ensure chain of custody
- Your N95 respirators are exposed to the vapor phase hydrogen peroxide process
- After processing, your N95 respirators are verified to ensure they are free of decontaminant

- The logistics provider returns your decontaminated N95 respirators back to your facility

