



# California Medical Device Recall Information



## Recall Name

**Dräger Expands Recall of PS500 Battery Power Supply  
Used With Evita V500 and Babylog VN500 Ventilators  
Due to Software Issue Causing Short Battery Run Time**

Recall Date	Product Description	Recalling Firm	Recall Reason
<p>Initial: 7/13/15</p> <p>Expanded: 12/1/15 and 2/4/16</p>	<p>Optional <b>PS500 Battery Power Supply</b></p> <p><i>Used with:</i></p> <ul style="list-style-type: none"> <li>• Evita V500 Ventilator</li> <li>• Babylog VN500 Ventilator</li> </ul>	<p><b>Dräger Medical, Inc.</b> Telford, PA</p>	<p><i>A software issue causes shorter than expected battery run times.</i></p> <p><i>The software issue with the power supply also prevents the appropriate alarm from sounding 5 minutes before the battery runs out of power and the device shuts down.</i></p>
Recall Class	Product Identification	Distribution	Affected Dates
I	<p><b>All Units Affected</b></p> <p>Catalog Numbers: <b>8416400 - 8417400</b></p>	CA, nationwide	<p>Manufactured: <b>June 1, 2011 to December 1, 2015</b></p> <p>Distributed: <b>June 1, 2011 to January 31, 2016</b></p>

FOR ADDITIONAL INFORMATION, PLEASE VISIT:

<http://www.fda.gov/MedicalDevices/Safety/ListofRecalls/ucm489108.htm>